**Instructions for Use**

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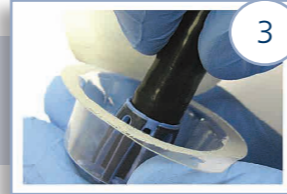
**Use**

Do not use lubricants to fit because of risk of dislocation of the device from the scope during the procedure.

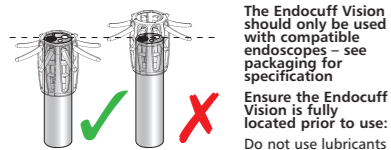
**Contraindications**

The Endocuff Vision is not intended for deep ileal intubation

The Endocuff Vision should not be used in cases with acute, severe colitis or where there is known colonic stricture.

The Endocuff Vision should or shall not be used for complex sub-mucosal dissection where a separate distal attachment is required.

**Warnings & Cautions**

of dislocation of the device from the scope during the procedure.

Always ensure the scope end is in good condition and not damaged before fitting the Arc device to the endoscope.

After use, the Endocuff Vision should be removed from the endoscope.

There is potential for biological hazards and therefore disposed of as clinical waste.

The Endocuff Vision is for single use only.

The Endocuff Vision is a sterile product.

Do not use if packaging is damaged.

Ensure that the patient's physical condition is not contraindicated for this procedure and do not forcibly operate an endoscope with an attached Endocuff Vision due to the potential risk of petechial scratching.


Be aware that the Endocuff Vision could detach from the endoscope during use if improperly attached. Be prepared to retrieve the device if this occurs.

If the body of Endocuff Vision is seen in the endoscopic view, stop the procedure immediately and withdraw with caution to minimize the risk of detachment.

**Labels**

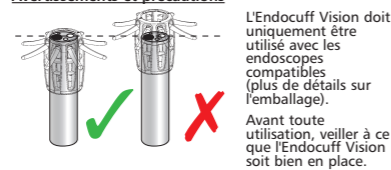	Single-use only
	Keep away from sunlight
	Batch code
	Sterilised by irradiation
	Do not use if packaging is damaged
	Biological risks
	Use by

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